

Surgical Incision Film

THERUPTOR™ FILM

Transparent Film Dressing

INSTRUCTIONS FOR USE

STERILE R

Sterilized using Irradiation (Gamma Irradiation)

MD

Medical Device



Do not use if package is damaged and consult instructions for use



Do not re-use



Keep dry

PRODUCT DESCRIPTION

Theruptor Film Dressing is a waterproof, sterile barrier for external contaminants and partials including liquids, bacteria and viruses. It is Latex Free and Hypoallergenic. It can be used to maintain a moist environment, and to secure devices to the skin.

INDICATIONS

Theruptor film dressing is a mechanical barrier to facilitate sterile protection to skin surrounding a surgical incision and to maintain surgical sterile environment. It can be used to cover the wound as barrier protection.

PRECAUTIONS

It is intended for external use only

- During Application don't stretch the dressing. Mechanical skin trauma may result if the dressing is applied with tension.

- This should not be re-sterilized by gamma, E-beam or steam methods.
- Antimicrobial ointments containing polyethylene glycols may affect the strength of film dressing.

DIRECTIONS FOR USE

- Open the Incision film dressing package and remove sterile dressing.
- Peel liner from the film dressing, exposing the adhesive surface.
- Position and middle the film dressing over the incision site, with adhesive side facing downward.
- Gently press the film dressing in place, smoothing from the middle to outwards.
- Slowly remove the outer liner while smoothing the film dressing border and applying firm pressure.
- Avoid stretching the film dressing on the skin.
- Conform and smooth the film dressing, gently pressing down on the edges to ensure optimal adhesion.

REMOVAL OF THE DRESSING

- Carefully lift the film dressing edges from the skin
- Continue to lift the border of the film dressing


SAFETY FACTOR

- Transparent film dressings allow easy site assessment. Inspect the site frequently for early signs of complications.
- Edge lift is not necessarily failure unless there is a channel from the edge of the film dressing to the incision site or wound.
- Change the film dressing according to your institution's protocol.
- For higher barrier protection, must maintain good adhesion on the total periphery of the site or wound, and be free of punctures.


STORAGE CONDITION

Store in a dry place.


INSTRUCTION FOR USE

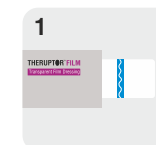


Remove the dressing from sterile package.

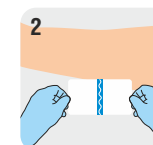


Make sure that the dressing size is bigger than the incision area to get complete coverage.

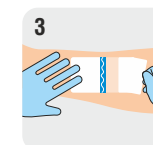


Pull the release paper and apply one half of the dressing.

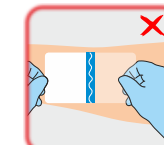


Do not touch the adhesive part of the film.

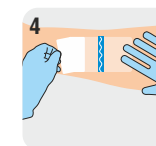

Without touching the dressing, pull the second release paper.

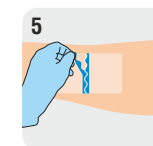

Remove the outer liner from the dressing.

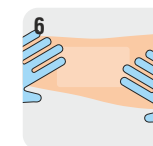

Ensure the edges are adhered to the skin, pressing it gently.



Ensure adhesive film is applied without any channeling.

Available Size: 100 x 120mm.

Marketed by : **Healthium Medtech Limited**

472- D, 13th Cross, 4th Phase, Peenya Industrial Area, Bangalore - 560 058, India.

Tel: +91 80 4186 8000 | Customer Care No. 080-41868198.

Consumer Care Address : Same As Above

Email : care@healthiummedtech.com

www.healthiummedtech.com

Manufactured by : **CareNow Medical Private Limited,**

#3/272-5, Neelambur Rd, Muthugoundenpudur, Coimbatore - 641402, TN, India.